GB/EE/LV/LT/RU/UA/MD-RO/GE/GR

corn-removal Medrull®

(GB) HELPS TO REMOVE CORNS. SOFT. PROTECTIVE. SMOOTHING ACTION.

CORN REMOVAL PLASTERS N2, N6, N12 effective against corns and calluses. Performs a protective and smoothing action, softening the skin and helps its mechanical removal, thus ensuring fast and pain-free corn removal.

DIRECTIONS FOR USE: Before applying, carefully clean and dry the area to be protected to ensure perfect adhesion and effectiveness. Remove the corn plaster from the paper, and position it centrally over the area to be protected. Apply a new corn plaster daily.

WARNINGS: Only use in the treatment of corns and calluses, for external use only. Do not use on broken or irritated skin. Avoid application in the presence of redness, cuts or cracks. Once sensitization or irritation occurs, immediately stop treatment. Diabetics, people with circulation problems of the blood, pregnant women, women who breastfeed, children younger than 16 years or in doubt, consult a doctor before using the product. Keep the product in its original packaging in a clean, dry, cool place. Keep out of reach of children. **Containing 35-40% salicylic acid.** Do not use for more than 2 weeks, unless discussed with the doctor. In case of serious incidents, report to the Manufacturer and Competent Authority. For waste management, please check the regulations in your area.

In the event of any serious incident occurred in relation to the device, please report to the Manufacturer and the relevant Competent Authority.

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Medrull® is a registered Trade Mark of FORANS International AG, Switzerland
Eurosirel S.p.A. Viale Europa, 30, 20047 Cusago (MI), Italy, eurosirel@eurosirel.com