## herpes patch

### HERPES PATCH. 15 pcs.

#### LIPSTICK USE PROOF. INVISIBLE. DISPOSABLE. INNOVATIVE NEW ADHESIVE.

**WHAT ARE THE SYMPTOMS?** Herpes is an infection roused by the presence of a virus, which appears as a cluster of skin blisters, quite often emerging on the lips: they are commonly known as cold sores.

**WHAT ARE THE SYMPTOMS?** At first, this type of Herpes appears as a warm, tingling sensation on a red spot on the lips. In a short time, within a few hours liquid-filled blisters start to appear. The appearance of cold sores may coincide with being undernourished, a fever, the beginning of menstruation, stress, an intense exposure to sunlight or after eating particular foods. Unfortunately, the infection caused by herpes labialis easily reappears because the virus that causes it survives inside some cells and is not eliminated even with the use of effective medicines. **PRECAUTIONS TO BE TAKEN AT ALL TIMES.** Avoid bringing your lips into contact with the bodies of other people, until the blisters have completely disappeared. Herpes labialis is highly contagious! Avoid very hot or cold foods because they may irritate the affected areas. **WHY USE THE "HERPES PATCH"?** The soft Polyurethane disc has a hypoallergenic adhesive that has been treated with Tea Tree Oil (lat. Melaleuca), known for centuries for its natural antiseptic properties. In this way, the main function of the patch is to provide protection from and towards the external environment, necessary to reduce the possibility of contagion, has been integrated with the secondary function of bacterial control of the herpes affecting microenvironment.

**INSTRUCTIONS FOR USE.** Wash your hands before applying the patch. Ensure that there are no traces of cream on the affected area and that skin is dry, so that the patch adheres well. Open the protective wrapper and pull out the patch lifting the opposite edge of the support as shown in the picture. Place half of the patch on the affected area and free the other half making it adhere completely on the herpes. Wear the patch until it comes off by itself. Continue with the application until the herpes disappears.

**WARNING.** Do not touch the patch with fingers to avoid infection of other parts of the body or other persons. Keep out of reach of children. This is a SINGLE USE product. In the event of any serious incident occurred in relation to the device, please report to the Manufacturer and the relevant Competent Authority.

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Medrull® is a registered Trade Mark of FORANS International AG, Switzerland
Eurosirel S.p.A. Viale Europa, 30, 20047 (Cusago) (MI), Italy, eurosirel@eurosirel.com