# Gyntonique

10 ovule de 2 grame

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# Gyntonique

10 pessaries of 2 g

based on hyaluronic acid, lactic acid and boric acid

**Medical device for vaginal administration.**

## Composition:

Semi-synthetic glycerides, hyaluronic acid sodium salt, boric acid, lactic acid, tocopheryl acetate, 18-beta-glycyrrhetic acid, tea tree oil.

## Property:

Gyntonique could be a valid non-pharmacological adjuvant for the prevention of bacterial and fungal vaginosis, as lactic acid allows the restoration and subsequent maintenance of the natural vaginal pH, avoiding the proliferation of pathogenic germs and favoring the colonization of the mucosa by the ones that are normally present in the vagina. Its action is enhanced by the presence of boric acid which, in addition to preserving the pessaries, contributes to the restoration of the normal vaginal pH.

Hyaluronic acid promotes the natural healing processes of the vaginal epithelium and making a mechanical barrier.

## Indications:

It is indicated to favor the re-epithelialization processes of the vaginal mucosa, in the prevention of vaginal issue based on bacterial and fungal origin, and as an adjuvant in their treatment.

## Dosage and method of use:

Introduce 1 pessary into the vagina in the evening for 10 days consecutive, unless otherwise indicated by the doctor. Take a pessaries from the blister, separate the two edges of the blister, extract the ovule and insert it deeply into the vagina, after a thorough intimate cleansing.

## Contraindications:

Known hypersensitivity of the user or of the sexual partner to the components. Do not use during pregnancy and lactation. The use for long periods must be limited to cases of real necessity and under medical supervision. As condom compatibility has not been tested, do not use before having sex. Do not use in patients under the age of 18. Do not use during concomitant topical therapies.

## Undesirable Effects:

Not known. In case of undesirable reactions, stop administration of the product.

## Warnings:

Do not use the product after the expiry date indicated on the package.  
Do not use the product if the package is not intact. Keep out of the reach and sight of children.  
Do not swallow. Do not use beyond the recommended time period The product must be used only vaginally, do not use in administration routes other than those indicated.

## Storage:

Store at a temperature not exceeding 25°C, protected from light and humidity.

## Diposal:

Blister: PVC+PE COD: "other 7", plastic. Cartonbox: COD: PAP 21, paper.

Leaflet: COD: PAP 22, paper.

Rev. 01 of 10/04/2021

INNATE s.r.l. Viale Industria, 11/13 – 15067 Novi Ligure (AL) – ITALIA  
Info@innate.it

ICIM S.p.a. Piazza Don Enrico Mapelli, 75 – 20099 Sesto San Giovanni (MI) – Country: ITALY