# KAdermin GYN

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# KAdermin GYN

## Single-use protective intimate gel

**INTENDED USE:** KAdermin GYN is an intimate gel with a moisturising, protective action, indicated for relieving discomfort such as itching, redness and burning. Thanks to the combination of moisturising and film-forming ingredients, it helps to protect and restore physiological balance in the event of excessive development of local microbial flora and to prevent irritation.

**INSTRUCTIONS:** Before use, gently cleanse the outer mucous membranes without rubbing; dry thoroughly and wash hands.

- Remove the cannula cap from the applicator.
- Gently squeeze the tube to dispense some product.
- Apply a sufficient amount to the affected areas.

The applicator is disposable. Apply the gel once a day, preferably at night before going to bed, for six consecutive days or according to medical advice.

### WARNINGS:

- Do not use the product for applications other than the ones indicated in the package leaflet.
- Do not use in the case of injuries and/or ulcers of the vaginal mucosa.
- Keep out of reach of children.
- The product is not intended for paediatric use.
- Do not use if the packaging has been opened or is damaged.
- Do not sterilise.
- Discontinue treatment if abnormal skin reactions occur; inform your doctor and the manufacturer.
- In the event of use beyond 7-10 days, consult a doctor. The treatment must not exceed 30 days.
- The medical device is not a spermicidal or contraceptive product.

**CONTRAINDICATIONS:** Overdose phenomena are not known. Established subjective hypersensitivity to the ingredients. There are no known interactions with drugs or medicinal substances. In the event of undesirable effects, inform the manufacturer immediately.

**STORAGE CONDITIONS:** Do not use after the expiry date indicated on the packaging; the expiry date refers to the product in undamaged packaging, correctly stored. Keep at room temperature, avoiding exposure to direct sources of light and heat and/or excessive ambient humidity. Avoid contact with water.

**INGREDIENTS:** Aqua, Glycerin, Sodium Hyaluronate, Threalose, Lactic Acid, Aloe Barbadensis leaf juice, PVP, Hydroxypropyl Guar, Propylene Glycole, Didecylidimonium Chloride, Isopropyl Alcohol, Sodium Silver Mercaptobenzimidazole Sulfonate.

### PACKAGING:

Six disposable vaginal applicators (5 ml)

Rev. 1 date 06/02/2024

Tested for Nickel, Chrome and Cobalt. Paraben-free.

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