# MASTER-AID® SPORT GHIACCIO SPRAY

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EN - ICE SPRAY has an immediate effect on bruises and sprains

**Intended use:** Ice spray provides immediate relief thanks to extreme cold instant action if sprayed on bruises and sprains.  
**Instructions for use:** Spray on the injured area, 25 cm away, for no longer than 5 seconds. Do not apply on abrasions, wounds and mucosa. Wounds caused by cold may occur if sprayed for a longer time and from a shorter distance. Use immediately after accident. Do not extend the use. Do not spray into eyes. In case of serious accidents related to the use of medical ice spray device, report it to the manufacturer who will notify competent Authorities. Avoid using in pediatric age (before 14 years).

**DANGER:** Extremely flammable aerosol. Pressurised container: May burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children. Do not spray on an open flame or other ignition source.

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REF 383.20

Does not contain ozone-hazardous propellants

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PIETRASANTA PHARMA S.p.A.