marimer

baby

EN 20 disposable filters + 2 nozzles

To be used with baby nasal aspirator MARIMER

[A] TIP

The shape and soft texture of the nasal tip ensures maximum comfort for your baby when inserting it into his or her nose.

[C] FILTER

The filter of the MARIMER nasal aspirator is equipped with a tab so it is easy to change it without getting your hands dirty. The MARIMER filtration technology ensures effective cleaning thanks to its high suction capacity. Mucus is sucked out without forcing and stays trapped in the bulb so it can't flow back into your baby's nose.

Precautions for use

Carefully read the instructions for use written on your nasal aspirator's case. Replace the tip at the first sign of wear and tear. The filters are single-use. Do not reuse them due to the risk of loss of performance and bacterial contamination. Do not boil. Consult your chemist. **KEEP OUT OF SIGHT AND OUT OF REACH OF CHILDREN.**

Warning

If symptoms persist or worsen, consult your doctor. Please report any serious incident related to this device to Laboratoires Gilbert - website: www.labogilbert.fr and/or the competent national authority.

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