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Ingrediente: Polidimetilsiloxani, siloxani, alchilmethyl silicon

[EN] Consumer information

Stratamark is a rapidly drying, non-sticky, transparent gel formulation for the prevention and treatment of Striae Distensae (stretch marks).

Stretch marks are a type of thin, flattened scar. In the early stages, the stretch mark appears slightly raised with a pink to red colour, which over time develops a slight purple tinge and then eventually becomes white and depressed with finely wrinkled lines.

Stratamark was developed for the prevention and treatment of all types of stretch marks, particularly those covering a wide surface area, resulting from pregnancy, adolescent growth spurts, weight gain or loss, or body-building.

When used as directed, Stratamark dries to form a thin, flexible and protective gel sheet. It creates a protective layer that is gas permeable and waterproof which hydrates and protects stretch marks, allowing the skin to normalise the collagen synthesis cycle.

Stratamark is used to soften and flatten stretch marks, reduce redness and discolouration associated with stretch marks as

well as to relieve the itching and discomfort of stretch marks.

Stratamark was developed to spread easily over large surfaces of the body (i.e. buttocks, abdomen, thighs, breasts and back), as well as exposed and sensitive areas.

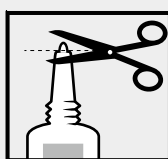
Intended use

Stratamark is used for the prevention and treatment of Striae Distensae (stretch marks). Stratamark does not penetrate below the level of the stratum corneum (the top layer of the skin).

Stratamark contains no alcohol, parabens or fragrances and is transparent and odourless. Stratamark is suitable for pregnant women, breastfeeding mothers, children, and people with sensitive skin.

Directions for use

On first use of 20 g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 50 g tubes remove the cap, then the protective seal and close with the cap after use.

Ensure that the skin is clean and dry. Apply a very thin layer of Stratamark to the relevant areas with the fingertips and distribute evenly to form a very thin layer and allow the gel to dry. A little amount of gel goes a long way.

When applied correctly, Stratamark should be dry in 5–6 minutes. If it takes longer to dry you have probably applied too much. Gently remove any excess gel with a tissue or paper towel and allow the drying process to continue.

Once dry, Stratamark can be covered with undergarments, sunscreens or cosmetics.

Stratamark should be applied once per day, or after each wash. For best results Stratamark should be maintained in continuous contact with the skin (24 hours a day / 7 days a week).

Removal of Stratamark

Stratamark can be easily removed from the hands after an application by washing them with warm water and soap. Repeat if necessary.

Recommended duration of treatment

Prevention of stretch marks during pregnancy:

Stratamark is recommended to use from the beginning of the second trimester (13 weeks of pregnancy) or at the first sign of stretch marks. The treatment should be continued following birth for a minimum of 60 to 90 days.

Prevention of stretch marks during growth spurts:

Stratamark should be used for the duration of the growth spurt or until the skin is no longer tightly stretched. The treatment should then be continued for an additional 60 to 90 days.

Treatment of existing stretch marks:

Stratamark should be used for at least 60 to 90 days (24 hours a day / 7 days a week). Continued use is recommended until no further improvement is seen. Severe and/or older stretch marks may need longer treatment.

Stratamark may also be used in combination with other stretch mark treatments (including laser therapy) with the advice of a physician.

How much Stratamark do I need?

Stratamark gel is a unique formulation that requires substantially less product per application than typical moisturising creams or cosmetic preparations.

- **One Stratamark 20 g (0.7 oz)** tube contains enough gel for approximately one month treatment for women in the late stages of pregnancy.
- **One Stratamark 50 g (1.75 oz)** tube contains enough gel for approximately two months treatment for women in the late stages of pregnancy.

In the beginning of pregnancy or after delivery one tube will last longer due to the smaller application area.

Caution

- For external use only.
- Stratamark should not be applied to third degree burns or to open wounds.
- Stratamark should not be placed in contact with mucous membranes or the eyes.
- Stratamark should not be applied over other skin treatments without the advice of your physician.
- Stratamark may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- If irritation occurs, discontinue use and consult your physician.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.
- Stratamark does not require special disposal methods. Please follow your local disposal regulations.
- Within the European Union, serious incidents related to the use of Stratamark should be reported to Stratpharma and the respective competent authority in the country the user is established.

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Stratpharma AG
Freyr Life Sciences GmbH
Stratpharma Limited