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Ingredients: Polidimetilsiloxani, siloxani, alchilmethyl silicon

[EN] Consumer information

A full contact flexible wound dressing for the management of dry skin and cutaneous rashes.

StrataCTX is a semi-occlusive, self-drying and non-sticky gel. StrataCTX may be directly applied to dry skin, open wounds and compromised or desquamating skin surfaces, including cutaneous rashes.

StrataCTX gel is bacteriostatic and inert. StrataCTX contains no steroids, alcohol, parabens or fragrances. StrataCTX is transparent allowing users to monitor therapy progress without removal of the dressing.

When used as directed StrataCTX dries to form a protective layer that is gas permeable and waterproof which hydrates and protects compromised skin areas and superficial wounds from chemical and microbial invasion.

How StrataCTX works

StrataCTX dries to form a thin, flexible wound dressing which hydrates and protects affected areas.

StrataCTX promotes a moist healing environment for wounds or compromised skin. This moist wound healing environment promotes faster re-epithelialisation and significantly reduces the skin's acute inflammatory response. StrataCTX lightly bonds to the most superficial damaged skin layer.

StrataCTX is ideal for large surface areas and contoured skin like head, face, hand and foot, as well as joints and hairy areas without the need for shaving.

Intended use

StrataCTX is used to relieve low grade inflammatory changes such as dry, itching, flaking, peeling and irritated skin.

For more severe inflammatory changes, StrataCTX reduces pain, redness and heat sensation.

StrataCTX was developed for use on all types of wounds, toxic and compromised skin including:

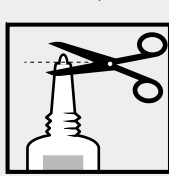
- Cutaneous reactions
- Pruritic, itchy skin
- Xerotic, dry skin
- Desquamation
- Fissures of skin and nail folds
- Blisters

- Medical Adhesive-Related Skin Injuries (MARS)
- Erythema
- Infusion reactions
- Rashes, including:
 - Maculopapular rash, hand-foot syndrome, GVHD, acneiform reaction, peri – and appendageal (hair follicles, sweat glands)

StrataCTX can be used with or without a secondary protective dressing. StrataCTX is suitable for children and people with sensitive skin. StrataCTX is intended for single patient use.

Directions for use

On first use of 10g and 20g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 50g tubes remove the cap, then the protective seal and close with the cap after use.

Ensure that the affected superficial area is clean and dry. Apply a very thin layer of StrataCTX directly to the affected area and allow the gel to dry.

When applied correctly to exposed areas, StrataCTX should be dry in 5–6 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a clean tissue or gauze and allow the drying process to continue.

Once dry, StrataCTX may be covered by sunscreen, cosmetics and clothing.

StrataCTX should be applied at least twice daily to affected areas, as needed or as advised by your physician.

For best results StrataCTX should be maintained in continuous contact with the skin (24 hours a day/7 days a week).

Recommended duration of treatment

StrataCTX is recommended as treatment following the first signs or symptoms on the skin and should be applied until resolved (24 hours a day/7 days a week) or until no further improvement is seen.

How much StrataCTX do I need?

StrataCTX gel is an advanced formulation that requires substantially less product per application than typical moisturising creams or gels. A little bit goes a long way.

- **StrataCTX 10g** is enough to treat an area of 36×15 cm twice per day for over 5 days.
- **StrataCTX 20g** is enough to treat an area of 36×15 cm twice per day for over 10 days.
- **StrataCTX 50g** is enough to treat an area of 36×15 cm twice per day for over 25 days.

Caution

- For external use only.
- StrataCTX should not be placed in contact with the eyes.
- StrataCTX should not be applied over topical medications unless advised by your physician.
- StrataCTX may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- Should your cutaneous reaction show signs of infection or failure to heal, immediately consult your physician.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for highly exudative wounds, tunneling wounds or 3rd degree burns.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.
- Do not use if the tube is damaged.
- StrataCTX does not require special disposal methods. Please follow your local disposal regulations.
- Within the European Union, serious incidents related to the use of StrataCTX should be reported to Stratpharma and the respective competent authority in the country the user is established.

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