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**Ingrediente:** Polidimetilsiloxani, Siloxani, Alchilmetil Siliconi

## [EN] Consumer Information

**Strataderm is a rapidly drying, non-sticky, transparent silicone gel formulation for the treatment of scars, both old and new.**

When used as directed Strataderm dries to form a silicone gel sheet, a protective layer that is gas permeable and waterproof which hydrates and protects scars from chemical and microbial invasion, allowing scars to normalise the collagen synthesis cycles, producing a normal mature scar.

Strataderm is used to soften and flatten scars, relieve the itching and discomfort as well as to reduce redness and discolouration associated with scars.

Strataderm can be easily applied to all areas of the skin, including exposed areas like the face and neck as well as joints and hairy areas without the need for shaving.

Strataderm was developed for the use on all types of scars including those resulting from

- general and cosmetic surgery
- trauma
- chronic wounds
- burns
- bites
- acne
- and other diseases of the skin.

The Updated International Clinical Recommendations on Scar Management confirmed that silicone gel should be used as first line therapy in the initial management of scars and in the prevention of hypertrophic scars and keloids.\*

## Intended use

Strataderm is used for the prevention and treatment of keloids and hypertrophic scars.

Strataderm can also be used in conjunction with other invasive scar treatment options to improve overall results, including surgical excision and resuturing, intralesional injections and pressure garments.

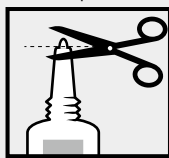
Strataderm contains no alcohol, parabens or fragrances.

Strataderm is suitable for children and people with sensitive skin.

Strataderm may be used on infants, during pregnancy and while breastfeeding.

## Directions for use

On first use of 5 g, 10 g and 20 g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 50 g tubes remove the cap, then the protective seal and close with the cap after use.

Ensure that the scar is clean and dry.

Apply a very thin layer of Strataderm to the scar and allow to dry.

When applied correctly, Strataderm should be dry in 3–4 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a tissue and allow the drying process to continue.

Once dry, Strataderm can be covered with sunscreen, cosmetics, pressure garments or casts.

Strataderm should be applied once daily, or twice daily to exposed areas or as advised by your physician.

For best results Strataderm should maintain continuous contact with the skin (24 hours a day / 7 days a week).

## Recommended duration of treatment

Strataderm is recommended as a minimum treatment of 60–90 days (24 hours a day / 7 days a week). Larger and older scars may take longer.

Continued use is recommended until no further improvement is seen.

## How much Strataderm do I need?

Strataderm is an advanced formulation that requires less product per application than typical creams or gels.

- **Strataderm 5 g (0.17 oz)** is enough to treat a 2–3 cm (1–2 inch) linear scar for over 90 days.
- **Strataderm 10 g (0.35 oz)** is enough to treat a 5–7 cm (2–3 inch) linear scar for over 90 days.
- **Strataderm 20 g (0.7 oz)** is enough to treat a 10–14 cm (4–6 inch) linear scar for over 90 days.
- **Strataderm 50 g (1.75 oz)** is enough to treat an area of 6×12 cm (2.5×5 inch) for over 90 days.

## Caution

- For external use only.
- Strataderm should not be applied on third degree burns or to open wounds.
- Strataderm should not be placed in contact with mucous membranes or the eyes.
- Strataderm should not be applied over other skin treatments without the advice of your physician.
- Strataderm may stain clothing if not completely dry.  
If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- If irritation occurs, discontinue use and consult your physician.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.

- Strataderm does not require special disposal methods. Please follow your local disposal regulations.
- Within the European Union, serious incidents related to the use of Strataderm should be reported to Stratpharma and the respective competent authority in the country the user is established.

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Freyr Life Sciences GmbH
Stratpharma Limited